The lot-specific document is available at the SERO website

## EN

### Product description

Autonorm™ Human Liquid is a liquid human-based control serum produced from blood collected from voluntary blood donors. Each unit is separately checked using CE approved tests to be negative for HBs antigen, HIV p24-antigen and HIV I, II and HCV antibodies. However, since no method can completely exclude the presence of infectious agents, this material should be handled as an ordinary patient sample.

Safety Data Sheet (SDS) is available at our website [www.sero.no](http://www.sero.no).

The lot-specific document which includes the analytical data and stability information is available online. Please visit [www.sero.no/theserolibrary](http://www.sero.no/theserolibrary) to download the lot-specific document. The 8-digit number printed on the label at the top of the first page of this document is the code required to access the lot-specific document on the website. If you do not have internet access or the label is missing, please contact your local distributor or SERO directly. SERO's contact information can be found on the last page of this document.

### Intended use

Autonorm™ Human Liquid is intended for in vitro diagnostic use as an assayed quality control material to verify precision of laboratory measurement procedures.

### Instructions for use •

To secure uniformity, ensure that each vial is similarly treated with respect to preparation for use. Always mix contents thoroughly before use.

1. Thaw the vial contents by gentle continuous rolling for approximately 40 minutes until the material has thawed completely.
2. Remove the amount of material required for testing and analyze immediately.
3. Prior to subsequent use, remove the vial from 2-8 °C and mix gently before sampling. Replace the stopper immediately and store the vial at 2-8 °C.
4. Any material removed from the vial but not used should be discarded. DO NOT RETURN control material to the original vial once removed.
5. If the material is to be used for analysis of trace elements, transfer the reconstituted material to a screw cap plastic tube. This is important to prevent contamination from the vial and the rubber stopper.
6. Establish target values and analytical ranges for the control material based on the measurement procedures in the lab.

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