The lot-specific
document is
available at the
SERO website

EN

Product description

Seronorm™ CRP Liquid is a liquid human-based control serum produced from blood collected from voluntary blood donors. Each unit is separately checked using CE approved tests to be negative for HBs antigen, HIV p24-antigen and HIV I, II and HCV antibodies. However, since no method can completely exclude the presence of infectious agents, this material should be handled as an ordinary patient sample.

Safety Data Sheet (SDS) is available at our website www.sero.no.

Use of this product as a trueness control is only recommended together with the lot-specific document which includes the analytical data and stability information. Please visit www.sero.no/theserolibrary to download the lot-specific document. The 8-digit number printed on the label at the top of the first page of this document is the code required to access the lot-specific document on the website. If you do not have internet access or the label is missing, please contact your local distributor or SERO directly. SERO's contact information can be found on the last page of this document.

Intended use

Seronorm™ CRP Liquid is intended for use as an assayed quality control material to monitor the precision and trueness of laboratory measurement procedures.

Instructions for use

1. Swirl the contents gently to ensure homogeneity and the material has achieved room temperature.
2. Remove the amount of material required for testing.
3. Promptly replace the stopper in the vial and store the remaining control material at 2-8 °C.
4. Any material removed from the vial but not used should be discarded. DO NOT RETURN control material to the original vial once removed.
5. Establish target values and analytical ranges for the control material based on the measurement procedures in the lab

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